

Boston ADVANCE® Cleaner

For Fluoro Silicone Acrylate and Silicone Acrylate Rigid Gas
Permeable Contact Lenses

DIRECTIONS FOR USE:

1. Wash hands with mild soap.
 2. Rub both sides of the lenses carefully with two to four (4) drops of Boston ADVANCE® Cleaner in the palm of your hand for twenty (20) seconds.
 3. Remove all traces of the cleaner by thoroughly rinsing with fresh tap water.
 4. Place lenses in empty lens case and fill with fresh Boston ADVANCE® Comfort Formula Conditioning Solution. **Soak lenses for at least four (4) hours (or overnight) before wearing.**
 5. Discard solution ninety (90) days after opening. Record date opened in space provided on bottle label. Note: When used daily, this solution will be depleted before ninety (90) days.
- For weekly protein removal, use Boston® ONE STEP Liquid Enzymatic Cleaner.
 - Lens cases should be replaced frequently to avoid potential microbial contamination and ocular irritation.

IMPORTANT: Please read carefully and keep this information for future use.

Description:

Boston ADVANCE® Cleaner is a sterile, concentrated, homogeneous surfactant solution containing alkyl ether sulfate, ethoxylated alkyl phenol, tri-quaternary cocoa-based phospholipid and silica gel as cleaning agents; with titanium dioxide.

Actions:

Boston ADVANCE® Cleaner loosens and removes lipids, stubborn deposits and debris which can impair the wearing comfort of your lenses.

Indications (Uses):

Boston ADVANCE® Cleaner is indicated for use to clean fluoro silicone acrylate and silicone acrylate rigid gas permeable contact lenses after each removal and before conditioning (wetting, soaking, disinfecting).

Contraindications (Reasons not to use):

Do not use this solution if you are allergic to any ingredient in this product. Not For Use With Soft (Hydrophilic) Contact Lenses.

TO HELP ENSURE PROPER USE OF THIS PRODUCT, BOSTON CLEANER BOTTLES HAVE A RED TIP TO INDICATE THAT THIS PRODUCT SHOULD NOT BE PUT INTO THE EYES.

WARNINGS: This cleaner will irritate your eyes unless removed from lenses by thorough rinsing. **IF INSTILLED INTO THE EYES, IMMEDIATELY REMOVE LENSES AND FLUSH EYES WITH FRESH TAP WATER UNTIL CLEANER IS THOROUGHLY REMOVED FROM THE EYES.**

To avoid contamination, never touch dropper tip of the container with your hands or to any surface. Keep bottle tightly closed when not in use. Contact lenses and solutions, if not handled properly can become contaminated by common bacteria known to cause ocular infections. Serious damage to the eye and loss of vision may result from using contaminated solutions or from wearing contaminated or damaged contact lenses.

PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYES. It is essential that you follow your eye care professional's directions and all labeling instructions for proper use and care of your lenses and lens care products, including the lens case.

EYE PROBLEMS, INCLUDING CORNEAL ULCERS, CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION. THEREFORE, IF YOU EXPERIENCE EYE DISCOMFORT, EXCESSIVE TEARING, VISION CHANGES, OR REDNESS OF THE EYE, IMMEDIATELY REMOVE YOUR LENSES AND PROMPTLY CONTACT YOUR EYE CARE PRACTITIONER.

Precautions:

To ensure proper hygiene and handling of contact lenses and contact lens solutions, always follow label directions carefully.

- Never reuse this solution.
- Always use fresh solution for soaking and storing lenses.
- Always wash your hands before handling your lenses.
- Store solutions at room temperature. Avoid freezing.
- After inserting your lenses, always thoroughly clean the interior of the lens case with hot water and air dry after each use.
- Use before expiration date on the container.
- **NEVER SOAK LENSES IN ANY CLEANER.**
- **ALWAYS RINSE LENSES THOROUGHLY WITH FRESH TAP WATER AFTER CLEANING.**
- **KEEP OUT OF REACH OF CHILDREN.**

Adverse Effects:

The following problems may occur:

- Eyes stinging, burning, itching (irritation)
- Excessive watering (tearing) of the eyes
- Unusual eye secretions
- Redness of the eyes
- Reduced sharpness of vision (visual acuity)
- Blurred vision
- Sensitivity to light (photophobia)
- Dry eyes

If you notice any of the above conditions, immediately remove and examine the lenses. If the problem stops and the lenses appear to be undamaged, thoroughly clean, rinse and disinfect the lenses and reinsert. If the problem continues or if a lens appears to be damaged, DO NOT

REINSERT. Contact your eye care professional immediately. If any of the above problems occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. Seek immediate professional identification of the problem and treatment to avoid serious eye damage.

How Supplied:

Boston ADVANCE Cleaner is supplied sterile in 1 fluid ounce (30mL), 2/3 fluid ounce (20mL) and 1/3 fluid ounce (10mL) bottles. All retail BOSTON® Solutions are packed in unit cartons. Bottles are sealed with an imprinted neckband and marked with a lot number and expiration date.

Do not use if imprinted neckband on bottle is missing or broken.

©Polymer Technology (USA) Corporation, a Bausch & Lomb Company

Made in USA 10/00

Manufactured for:

Polymer Technology (USA) Corporation,
a Bausch & Lomb Company
Rochester, NY 14603-0450 USA

Questions or Comments:

1-800-333-4730

www.bausch.com

3404915W