

Bausch & Lomb ReNu® MPS™ Multi-Purpose Solution

This Insert contains important information, please read it carefully and retain for future reference.

ReNu® MPS™ Multi-Purpose Solution cleans, rinses and disinfects, all steps in one bottle. ReNu® MPS™ Multi-Purpose Solution is proven safe and effective for soft contact lenses.

Unique, sterile formula, ReNu® MPS™ Multi-Purpose Solution has poloxamine and DYMED® for clean, comfortable lenses.

CONTENTS

A sterile solution that contains boric acid, edetate disodium, sodium borate and sodium chloride; DYMED® (Polyaminopropyl biguanide) 0.00005%, poloxamine 1%.

CONTAINS NO CHLORHEXIDINE AND NO THIMEROSAL.

DIRECTIONS: Always wash and rinse your hands before you handle your lenses. Lenses must be thoroughly cleaned, rinsed and soaked each time they are removed to achieve disinfection.

To thoroughly clean and disinfect your lenses, you must complete each of the following steps every time you remove them.

Step 1: Place at least three drops of **ReNu MPS** Multi-Purpose Solution on each side of the contact lens and rub for 20 seconds. No separate daily cleaner needed.

Step 2: Remove surface debris by rinsing thoroughly with **ReNu MPS** Multi-Purpose Solution.

Step 3: Place cleaned contact lenses in lens case and fill with fresh **ReNu MPS** Multi-Purpose Solution. **SOAK FOR AT LEAST FOUR HOURS** or overnight.

Store your lenses in the unopened lens case until ready to wear, up to a maximum of 30 days.

Remove your lenses from the lens case and place in your eyes. After each use pour the ReNu® MPS™ Multi-Purpose Solution out of the case, rinse with fresh ReNu® MPS™ Multi-Purpose Solution and allow to dry in the air.

WEEKLY CLEANING: Protein removal with an enzymatic cleaner may be necessary for some patients. Please consult your eye care practitioner.

PRECAUTIONS

- Lens care procedures recommended by your practitioner must be followed. Failure to follow these procedures may result in the development of serious eye infections.
- Discard the BAUSCH & LOMB ReNu® MPS™ Multi-Purpose Solution from the lens case after each use.
- Do not touch bottle tip to any surface or your eye since this may contaminate the solution or cause injury to the eye.
- Keep cap closed when not in use to avoid contamination or evaporation.
- Do not use BAUSCH & LOMB ReNu® MPS™ Multi-Purpose Solution with a heat disinfection method.
- Discard remaining solution 6 months after first opening.
- Do not use beyond the expiry date.
- Do not use any eye medication in conjunction with this solution unless under medical supervision.
- Keep out of the reach of children.

IMPORTANT

If irritation occurs, persists or increases, discontinue use and consult your eye care practitioner.

CE 0050



Bausch & Lomb Incorporated
Rochester NY 14609 USA



Bausch & Lomb Incorporated
106 London Road,
Kingston-upon-Thames KT2 6TN, UK

Made in Italy

BAUSCH & LOMB, ReNu MPS, DYMED and Because Your Eyes Are So Important are trademarks of Bausch & Lomb Incorporated.

© Bausch & Lomb Incorporated.

