

BAUSCH & LOMB

ReNu®

1 Step™ Daily Protein Remover Liquid

**PLEASE READ CAREFULLY AND KEEP THE PACKAGE
INSERT FOR FUTURE USE IN CASE YOU HAVE A PROBLEM.**

ReNu® 1 Step™ Daily Protein Remover is a simple and convenient way to effectively remove protein deposits from soft contact lenses on a daily basis. This product is especially formulated for daily use to simultaneously enzymatically clean and disinfect your lenses every time you soak your lenses in ReNu® Multi-Purpose Solution. Use with ReNu Multi-Purpose Solution for convenient enzymatic cleaning during disinfection of soft (hydrophilic) lenses.

DESCRIPTION:

ReNu 1 Step Daily Protein Remover is a sterile preservative-free solution that contains proteolytic enzyme (subtilisin), glycerin, and borate buffer.

ACTIONS:

While you are wearing your soft (hydrophilic) contact lenses they develop a surface film which is composed of protein and other substances found in your tears. **ReNu 1 Step Daily Protein Remover** contains an enzyme to help break up the deposits and facilitate their removal.

INDICATIONS (Uses):

ReNu 1 Step Daily Protein Remover is indicated for use with daily wear and extended wear soft (hydrophilic) contact lenses to simultaneously enzymatically clean lenses while they are being disinfected for a minimum of four hours in **ReNu Multi-Purpose Solution**. **Use as recommended by your eye care practitioner.**

CONTRAINDICATIONS (Reasons not to use):

If you are allergic to any ingredient in **ReNu 1 Step Daily Protein Remover** do not use this product.

WARNINGS:

PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE. It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use and care of your lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can

develop rapidly and lead to loss of vision. Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when these lenses are worn overnight.

Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your eye care practitioner. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement.

If you experience eye discomfort, excessive tearing, vision changes, or redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner. It is recommended that all contact lens wearers see their eye care practitioner twice a year or if directed, more frequently.

ReNu 1 Step Daily Protein Remover IS NOT FOR USE DIRECTLY IN THE EYE OR FOR INTERNAL USE.

This product has been developed for use with **ReNu Multi-Purpose Solution** for soft (hydrophilic) contact lenses. Its effectiveness has not been demonstrated with any other products. Use **ReNu 1 Step Daily Protein Remover** only with **ReNu Multi-Purpose Solution** for soft (hydrophilic) contact lenses.

PRECAUTIONS:

Always use fresh **ReNu Multi-Purpose Solution** for soaking lenses and discard immediately after use.

Keep the bottle tightly closed when not in use.

Use only **ReNu** brand lens cases, or a lens case for soft contact lenses that holds 4 ml in each side.

Lenses should never be placed directly in the eye from the cleaning/disinfecting solution. Always rinse lenses with **ReNu Multi-Purpose solution** after cleaning and disinfecting.

Never soak lenses in **ReNu 1 Step Daily Protein Remover** only. Never place more than one drop of **ReNu 1 Step Daily Protein Remover** in lens well.

NOTE: Failure to rinse your lenses may cause discomfort due to debris remaining on the lens.

To prevent contamination and to help avoid serious eye injury, after each use always empty and rinse lens case with fresh rinsing solution and allow to air dry. Replace your **ReNu** lens case every three months to avoid microbial contamination.

Special Storage Conditions:

Use before the expiration date printed on the carton and bottle.

Store at room temperature (15° - 30°C, 59° - 86°F).
Keep out of the reach of children.

Use of ReNu 1 Step Daily Protein Remover is not a substitute for cleaning by rubbing and rinsing. Always follow the directions for use with **ReNu Multi-Purpose Solution** and the instructions of your eye care practitioner.

ADVERSE REACTIONS (Problems and what to do):

The following problems may occur: Eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eyes, unusual eye secretions, redness of the eyes, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above, **IMMEDIATELY REMOVE AND EXAMINE YOUR LENSES.** If the problem stops and the lenses appear to be undamaged, thoroughly clean, rinse, and disinfect the lenses and reinsert them. Do not reinsert a damaged lens. If the problem continues, immediately remove your lenses and immediately consult your eye care practitioner.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization, or iritis may be present. Seek immediate professional identification of the problem and treatment to avoid serious eye damage.

DIRECTIONS FOR USE

Good Lens Care Practices:

Soft lenses must be cleaned and disinfected each time they are removed.

- Always wash and rinse your hands before handling your lenses. This will help to prevent eye infections by removing dirt and oils that could get on the lenses.
- To avoid mix-ups, always handle one lens, the right or left, first.
- Use only the solutions recommended by your eye care practitioner. Seek advice from your eye care practitioner before making any changes to your care regimen to ensure compatibility with lenses.
- Always follow the directions for use in the labeling included with each solution as instructions may be different for each solution.

DIRECTIONS FOR USE WITH ReNu MULTI-PURPOSE SOLUTION:

- 1) Fill both sides of your **ReNu** lens case with fresh **ReNu Multi-Purpose Solution**.
- 2) Add **ONE** drop of **ReNu 1 Step Daily Protein Remover** to each side of your lens case to enzymatically clean your lenses while they are soaking in **ReNu Multi-Purpose Solution**.
- 3) Remove your lenses and clean and rinse them according to the **ReNu Multi-Purpose Solution** directions for use, and then place each lens into the appropriate side of your lens case.
- 4) Place the caps on your lens case and shake gently to assure that mixing occurs.

- 5) Soak your lenses in the cleaning and disinfecting solution in your lens case overnight (a minimum of 4 hours). Do not exceed 12 hours in the **ReNu 1 Step Daily Protein Remover/ReNu Multi-Purpose Solution**.
- 6) When you are ready to wear your lenses, remove them from the cleaning and disinfecting solution in your lens case. Rinse lenses thoroughly with **ReNu Multi-Purpose Solution** to remove loosely adhering debris remaining from the cleaning process. Your lenses are now ready to wear.
- 7) Always discard solution, rinse your lens case after each use with **ReNu Multi-Purpose Solution**, and allow to air dry.
- 8) If you do not intend to wear your contact lenses immediately, store lenses in a lens carrying case containing fresh **ReNu Multi-Purpose solution**. For lens wearing comfort, use **ReNu 1 Step Daily Protein Remover** each time you remove your lenses for cleaning and disinfection. **Store at room temperature (15° - 30°C, 59° - 86°F).**

HOW SUPPLIED:

ReNu 1 Step Daily Protein Remover is supplied as a 30 day supply in a 7.5ml bottle containing 3ml of solution. **ReNu 1 Step Daily Protein Remover** is for use with daily and extended wear soft contact lenses. For use as recommended by your eye care practitioner.

ReNu Contact Lens Case

For storage of soft contact lenses during chemical disinfection, and simultaneous enzymatic cleaning/Chemical disinfection only. **DO NOT USE ENCLOSED CASE WITH HEAT.** Before first use, rinse lens case and caps thoroughly with either saline solution or disinfecting solution, depending on the method of disinfection used. **DO NOT USE SOAP, DETERGENTS OR OTHER HOUSEHOLD CLEANERS.**

PRODUCT SATISFACTION GUARANTEE

If you are not completely satisfied with this product, call the toll-free number.

Toll Free Product Information - Call: 1-800-553-5340

Bausch & Lomb, ReNu and 1 Step are trademarks of Bausch & Lomb Incorporated.

© Bausch & Lomb Incorporated

Manufactured by:

Bausch & Lomb, Rochester, NY 14603-0450

Made in USA.

www.bausch.com

6654200W