

PLEASE READ CAREFULLY AND KEEP THIS PACKAGE INSERT FOR FUTURE USE IN CASE YOU HAVE A PROBLEM.

**Bausch & Lomb**  
**ReNu**

MultiPlus®  
Multi-Purpose Solution

Cleans, Rinses, Disinfects, Stores and Removes Protein Daily.  
All Steps In One Bottle.

**DIRECTIONS:** To clean, disinfect and remove protein daily, complete these simple steps. This daily regimen is recommended by Bausch & Lomb for a healthy and comfortable contact lens wearing experience:

**STEP 1:**

Place 3 drops of ReNu MultiPlus Multi-Purpose Solution on each side of the lens surface and gently rub for 20 seconds.

**STEP 2:**

Thoroughly rinse each side of the lens for 5 seconds with ReNu MultiPlus Multi-Purpose Solution.

**STEP 3:**

Place cleaned contact lens in the lens case and fill with fresh ReNu MultiPlus Multi-Purpose Solution. Soak at least 4 hours. Remember to always use fresh solution – never reuse solution.

**Alternative no-rub regimen:** To clean, disinfect and remove protein daily without a rub step, complete Step 2 and Step 3 only, as described above.

Always follow your eye care professional's instructions. Based upon your individual tear chemistry and lens-wearing schedule, your eye care professional may recommend additional products or procedures.

**STORAGE:** You may store your lenses in the unopened lens case until ready to wear, up to a maximum of 30 days.

**CONTENTS:**

A sterile, isotonic solution that contains HYDRANATE® (hydroxyalkylphosphonate), boric acid, edetate disodium, poloxamine, sodium borate and sodium chloride; preserved with DYMED® (polyaminopropyl biguanide) 0.0001%.

**ACTIONS:**

Cleans, loosens and removes accumulations of film, protein, other deposits and debris from soft contact lenses. Removes protein most effectively when used daily. Destroys harmful micro-organisms on the surface of the lens. Rinses, stores and wets lenses before insertion. Formulated for use in disinfection and storing lenses for up to 30 days after disinfection.

**INDICATIONS (Uses):**

Bausch & Lomb ReNu MultiPlus Multi-Purpose Solution is indicated for use in the daily cleaning, removing protein deposits, rinsing, chemical (not heat) disinfection and storage of soft (hydrophilic) contact lenses as recommended by your eye care professional.

**CONTRAINDICATIONS (Reasons not to use):**

If you are allergic to any ingredient in this product, do not use.

**WARNINGS: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE.**

It is essential that you follow your eye care professional's directions and all labeling instructions for proper use and care of your lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when these lenses are worn overnight. Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your eye care professional. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and

disinfection or for disposal and replacement. Studies have also shown that smokers have a higher incidence of adverse reactions. If you experience eye discomfort, excessive tearing, vision changes, redness of the eye, immediately remove your lenses and promptly contact your eye care professional. It is recommended that contact lens wearers see their eye care professional twice each year, or if directed, more frequently.

- To avoid contamination, do not touch tip of container to any surface. Replace cap after using.
- Not for use with heat (thermal) disinfection.

**PRECAUTIONS:**

- Never re-use this solution.
- Keep the bottle tightly closed when not in use.
- Store solution at room temperature.
- Use before the expiration date marked on the carton and bottle.
- Keep out of reach of children.

**ADVERSE REACTIONS (Problems and what to do):**

The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above:

**IMMEDIATELY REMOVE YOUR LENSES.**

- If the discomfort or problem stops, then look closely at the lens.
- If the lens is in any way damaged, DO NOT put the lens back on your eye. Place the lens in the storage case and contact your eye care professional.
- If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, thoroughly clean, rinse and disinfect the lens, then reinsert it.
- If the problem continues, IMMEDIATELY remove the lens and consult your eye care professional.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. Seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage.

**GOOD LENS CARE PRACTICES:**

- Always wash and rinse your hands before you handle your lenses.
- Clean, rinse and disinfect your lenses each time you remove them.
- Always handle the same lens, the right or the left, first in order to avoid mix-ups.
- After use, always empty and rinse the lens case with fresh ReNu MultiPlus Multi-Purpose Solution, and allow to air dry.

**HOW SUPPLIED:** Bausch & Lomb ReNu MultiPlus Multi-Purpose Solution is available for purchase in sterile 4 Fl. Oz. (118ml), 8 Fl. Oz. (237ml), 10 Fl. Oz. (296ml), 12 Fl. Oz. (355ml) and 16 Fl. Oz. (473ml) plastic bottles. Bottles and cartons are marked with a lot number and expiration date.

**Toll Free Product Information**

Call: 1-800-553-5340

Bausch & Lomb, ReNu MultiPlus, HYDRANATE, DYMED and Because Your Eyes Are So Important are registered trademarks of Bausch & Lomb Incorporated.

© Bausch & Lomb Incorporated

U.S. Patent 5,858,937

Manufactured by: Bausch & Lomb, Rochester, NY 14609. Made in USA.