




## Monitor your vision with this Amsler Grid

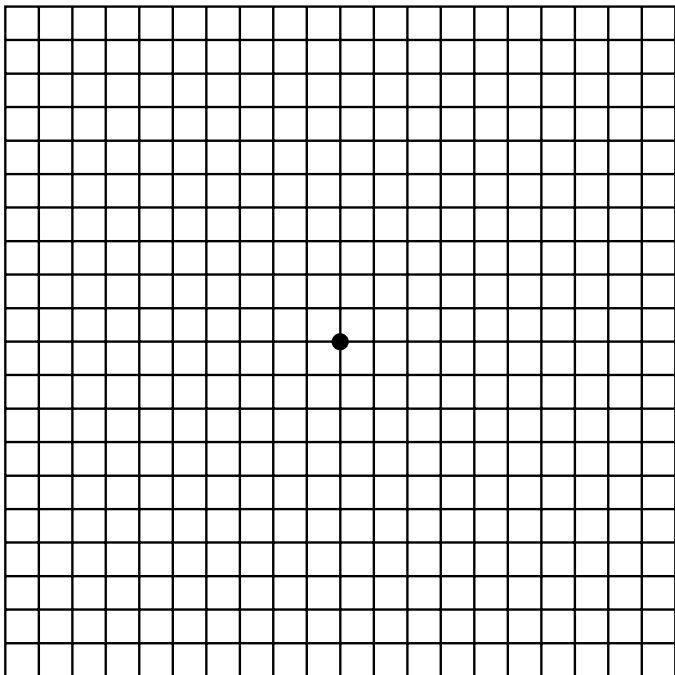

### To use the Amsler Grid:

- Wear your eyeglasses or contact lenses
- Hold grid at comfortable reading distance
- Cover one eye at a time
- Stare at center dot; do not let your eye drift

### Contact your eye doctor immediately if:

- A straight line appears wavy or bent
- A box differs in size or shape
- Any area is missing, blurry, or discolored



PreserVision is a trademark of Bausch & Lomb Incorporated or its affiliates.

© 2020 Bausch & Lomb Incorporated or its affiliates. PN09487 PVN.0016.USA.20